DONJOY®

Icestrap™

enovis™

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed healthcare professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS

The DonJoy Icestrap is designed to provide comfort and support to the abdomen and support to the hip. It may be suitable for post-operative care following abdominal and hip surgery. It allows localized external application of cold therapy and compression after surgery or injury, to help minimize inflammation, swelling, and pain.

- Post-operative care (abdominal or hip surgery)

PERFORMANCE CHARACTERISTICS

- Providing support and stabilization to the limb or body segment.
- Providing partial immobilization or controlled movement to the limb or body segment.
- Providing cold therapy for a temporary reduction of swelling and pain after injury.

CONTRAINDICATIONS

If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.

Do not use if you are allergic to any of the materials contained within this product.

FOR COLD THERAPY: Sickle-cell Anemia, Raynaud's or other vasospastic diseases, cold allergy, cold hypersensitivity, previous cold injury, or any compromised local circulation, decreased skin sensitivity, hypertension, paroxysmal cold hemoglobinuria, and cryoglobulinemia.

WARNINGS AND PRECAUTIONS

WARNING! THIS DEVICE CAN BE COLD ENOUGH TO CAUSE SERIOUS INJURY. SERIOUS ADVERSE REACTIONS AND SAFETY HAZARDS MAY OCCUR WHEN USING THIS DEVICE.

- We recommend that the first application be made under supervision of a healthcare professional.
- Cold products may cause injuries, e.g. burns, blisters, frostbite, swelling, discoloration and skin or tissue necrosis if improperly used or monitored.
- If you have skin conditions/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a healthcare professional.
- If you experience any discomfort, pain, swelling or sensation changes, or any unusual reactions while using this product, consult your healthcare professional immediately.
- This device can be cold enough to cause serious injury, including tissue necrosis.
- You must be able to check your skin condition under the Cold Pack frequently (at least every hour).
- DO NOT use this device if you cannot check your skin condition frequently (at least every hour).
- If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
- Check for increased pain, burning, numbness, tingling, increased redness, discoloration, itching, increased swelling, blisters, irritation or other changes in skin condition under the Cold Pack or around the treatment area.
- If you experience any of these conditions, immediately discontinue use of this device and contact your physician.
- This product is for single patient multiple use only.
- This product is not recommended for individuals with sensitive skin (e.g. elderly and children).
- Do not use this device if it is damaged and/or packaging has been opened.
- The product should be removed during long periods of rest.
- It is not recommended to drive while wearing the brace as your ability to brake may be impaired.
- Do not use during showering or bathing.
- ALWAYS use with a barrier/cloth between your skin and the Cold Pack (use the skin-protective textile).
- Keep the product away from children.
- All Cold Pack ingredients are non-toxic. They are, however, not intended for ingestion.
- If contents of the Cold Pack get in contact with eye or skin, rinse with water immediately.
- Do not heat the Cold Packs (microwave, oven, hot water, etc.).
- Do not use on open wound(s) and/or sensitive skin.
- Do not use this product in any way that may damage its integrity or use the device other than intended.
- Do not use this product if it is damaged or defective.
- Do not use this device if it was damaged and/or packaging has been opened.
- Do not use this product in any way that may damage its integrity.
- Do not sit or lie over the Cold Pack or fall asleep with the Cold Pack on the skin.
- Keep the product away from children.
- The frequency and duration of use should be determined by a healthcare professional.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

LIMITATIONS ON SPECIAL PATIENT POPULATIONS

Limit the use of this device with patients who are unresponsive, incapacitated, have altered mental status or altered pain perception, on analgesics or anesthetics, as well as patients taking hypnotics, anxiolytics, or antidepressants, must be monitored frequently during use of this device. These patients may not be able to perceive pain, burning, numbness, tingling or decreased sensation and may be susceptible to injury. Discontinue cold therapy immediately at the first sign of cold injury.

APPLICATION INSTRUCTIONS

The DonJoy Icestrap includes the following items:

- 1 abdominal belt
- 2 cold packs (the inner side faces the skin, the outer side has two hook fasteners for attachment to the belt)
- 1 skin-protective textile (to be applied between your skin and the cold pack)
- 1 thigh strap (with extension if required)
- 1 zip bag (you may use the bag to protect the cold pack when you place it in the freezer)

1. Place the cold pack in the freezer for at least 4 hours before use (domestic deep freeze temperature between -12°C & -20°C).

2a/b. Once the cold pack is cold/frozen, cover the inner side with the skin-protective textile using the elastic strings.

The side of the cold pack facing the skin is the one without any hook fasteners. Feel the cold pack and ensure the temperature is adequate before use.

WARNING! Be sure to place the skin-protective textile between your skin and the cold pack (See WARNING & PRECAUTIONS).

3a/b. Position the cold pack and its skin-protective textile with the outer side facing upwards. Guide the thigh strap through the loops. The set is now ready to be assembled on the belt.

4. Open the belt completely and place it on a flat surface in front of you. Fix the cold pack with its cover to the belt using the hook and loop fasteners. You may adjust the height as needed.

Notes:

- On the belt, the loop fabric is positioned on the central stays
- On the cold pack, the hook fabric should be positioned on the side without the protective cover

5. Apply the cold pack protected by its textile cover on the painful region. Close the belt by using the front closure brackets.

6. Attach the thigh strap to the bottom of the cold pack, wrap it around the thigh, and fix it using the hook and loop fasteners.

Note: If needed, you may enlarge the thigh strap using the extension.

7. Apply the cold therapy for 20 minutes or according to your professional healthcare recommendation.

CLEANING INSTRUCTIONS

- Belt, Skin-protective Textile and Thigh Strap: Machine washable at gentle cycle (max 40°C) with a mild detergent.
- Hook and loop of the straps should be closed for washing to avoid any damage.
- Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not iron.
- Cold Pack: may be cleaned with water, moist towel or sponge. Do not machine wash the cold packs.

DISPOSAL

Dispose of the device and its packaging in accordance with local or national environmental regulations.

COMPOSITION

- Belt: Polyester, Cotton, Polyamide, Steel, Polyethylene, Elastane, Polyurethane
- Skin-protective textile: Cotton, Elastane, Polyester
- Thigh Strap: Polyamide, Polyurethane, Elastane, Polyester
- Cold Pack:
 - Gel: Water, Monopropylene glycol, Dihydroxyethylcellulose, Triethanolamine, Sodium Chloride, Potassium substitute
 - Fabric: Polyester, Polyamid, Polyethylene, Polyurethane, Vinyl ethylene-acetate, Cotton
- Plastic Bag for Freezer: Polyethylene

NOT MADE WITH NATURAL RUBBER LATEX

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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DJO France SAS,
MDSS GmbH
MDSS CH GmbH