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2. Important safety information

Before you start using this product for pumping, please read your Medela Symphony® breast pump Instructions for Use for additional important safety information.

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorised representative and to your national authority. Warnings identify all Instructions for Use that are important for safety. Failure to observe these Instructions for Use can lead to personal injury or damage to the product. The following symbols and signal words show the significance of the warnings:

WARNING

Can lead to serious injury or death.

CAUTION

Can lead to minor injury.

NOTICE

Can lead to material damage (not related to personal injury).

Information

Useful or important information that is not related to safety.

WARNING

To avoid health risks and reduce the risk of severe injury:

- The Hands-free Collection Cups are a single-user product. Use by more than one person may present a health risk and voids the warranty.
- Clean all parts that come into contact with your breast and breast milk prior to first use and after every use (before milk can dry onto the device) according to the instruction in chapter 7.
- The tubing poses a risk of strangulation when played with and small parts may be swallowed by toddlers. Close supervision is necessary when the Hands-free Collection Cups are used in the vicinity of children.

CAUTION

To avoid fire or burns:

- The Hands-free Collection Cups and accessories are not heat-resistant. Keep away from heated surfaces or open flames.
- Do not use near flammable materials.

To avoid health risks and reduce the risk of injury:

- Before each use visually inspect the individual components for cracks, chips, tears, discoloration or deterioration. In the event that damage to the device is observed, please discontinue use until the parts have been replaced.
- This product cannot be serviced or repaired. Do not repair yourself. Do not modify the device.
- Never use a damaged device. Replace defective or worn parts.
- Use the Hands-free Collection Cups only for their intended use / purpose as described in these Instructions for Use.
- Always place the pump on a stable, non-slip surface. Avoid excessive movement that could increase the distance between the pump and the collection cups. This could cause the pump to be pulled off its surface by the tubing, leading to potential pump damage or minor injury.
- If tubing becomes moldy, discontinue use and replace tubing.
- Do not microwave or boil breast milk. Microwaving can cause severe burns to baby's mouth from hot spots that develop in the milk during microwaving (microwaving or boiling can also change the composition of breast milk).
- Wash and disinfect all parts that come into contact with your breast and breast milk prior to first use.
- Only use Medela Symphony® breast pump with the Hands-free Collection Cups described in this Instructions for Use.
- If pumping is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the breast shield with your finger and remove the breast shield from your breast.
- Choose the correct breast shield size for pumping. Using an unsuitable breast shield size may cause pain, discomfort and reduction in milk flow. Visit www.medela.com/fittingguide or see a lactation consultant/ breastfeeding specialist who can help you get a proper fit or if you can express only minimal or no milk or if expression is painful. See **chapter 8.1** for more information.

3. Intended use

3.1.1 Indications for use

The Hands-free Collection Cups reusable set is intended to be used by lactating women for expressing breast milk with the Medela's Symphony® breast pump. The pump set is a single-user product.

3.1.2 Contraindications

There are no known contraindications for the Hands-free Collection Cups.

4. Product description

Refer to figure  1 on Chapter 1.

For ordering information about accessories or replacement parts please see **chapter 11**.

All pictures shown in these Instructions for Use are for illustration purpose only. The actual product may vary due to local differences or product enhancement. Medela reserves the right to substitute any component or accessory with a replacement of equivalent performance.

5. Cleaning before first use

Before first use, it is important you do the following to ensure safety for you and your baby:

1. **Disassemble, wash and boil all parts of the collection cups** according to the instructions in **chapter 6** and **7**. Do **not** wash or boil the tubing and the Symphony® cap.
2. **When assembling, please ensure the cups are fully sealed until you hear a snap** - see **chapter 8.2**.

6. Disassembling your Hands-free Collection Cups

Refer to figure  2 on Chapter 1.

1.  2-01 Open the Collection Cups using the tab ( 2-01-01) at the bottom.
2.  2-02 Remove the yellow membrane from the breast shield.

7. Cleaning

It is important you do the following before using your Hands-free Collection Cups for the first time: Disassemble all parts and clean following cleaning Instructions for Use.

Supplies needed:

- Mild dish washing liquid
- Clean soft brush or sponge
- Clean wash basin or dishwasher
- Drinking-quality water
- Clean pot for boiling water

Parts to wash and disinfect:

- Breast shields
- Outer shells
- Yellow membranes

CAUTION

Disassemble and wash all parts that are exposed to breast milk immediately after use. This will help to remove breast milk residue and prevent growth of bacteria.

- Wash hands thoroughly with soap and water.
- Only use drinking-quality tap water or bottled water for washing and disinfecting.
- Do not place the Hands-free Collection Cups' parts directly in the sink to rinse or wash. Use a clean wash basin used only for infant items.
- Do not use a dish towel to rub or pat items dry.
- Do not use disinfectants for cleaning.
- Inspect the parts for any residue after dishwashing. Repeat the washing steps, if necessary.

NOTICE

Do not clean or rinse the tubing and the Symphony® cap under any circumstance.

- Inspect the tubing and the Symphony® cap before use. If you find condensation, breast milk and /or mould in the tubing and the Symphony® cap, discontinue use and replace the tubing and the Symphony® cap. Refer to chapter 11 for ordering information.
- Do not run the breast pump with wet tubing/Symphony® cap; doing so will cause damage to your breast pump.

NOTICE

Pay attention to the following:

- Take care not to damage parts of the Hands-free Collection Cups during cleaning.
- Store the dry Hands-free Collection Cups in a clean bag/container until next use.
- Do not store wet or damp parts as mold may develop.
- If you notice a white residue on parts after boiling, your water may have a high mineral content. Remove residue by wiping parts with a clean towel and allow to air dry.
- Distilled water is recommended when boiling parts to prevent substantial mineral build-up over time, which may compromise your parts.

7.1 Cleaning overview

Washing and disinfecting are two different activities. They must be done separately to protect you and your baby, and to maintain the performance of your breast pump.

Wash – To clean the surfaces of the parts by physically removing milk residues and other possible contaminants.

Disinfect – To efficiently lower the number of germs that may be present on the surfaces of the washed parts.

	Breast shield	Outer shell	Yellow membrane
When to clean			
Before first use	Disassemble the parts according to chapter 6 Wash the parts according to chapter 7.2 Disinfect the parts according to chapter 7.3		
After each use	Disassemble the parts according to chapter 6 Wash the parts according to chapter 7.2		
Once per day after use	Disassemble the parts according to chapter 6 Wash the parts according to chapter 7.2 Disinfect the parts according to chapter 7.3		

Do not clean or rinse the tubing and the Symphony® cap under any circumstances.

7.2 Washing

Refer to figure 3 on Chapter 1.

Rinse and wash by hand (before first use and after each use)

Do not place the parts directly in the kitchen sink for rinsing and washing. Use a dedicated wash basin for infant items.

1. Rinse the disassembled parts, except for the tubing and the Symphony® cap, with cold water (approx. 20 °C).
2. Wash these parts thoroughly with plenty of warm, soapy water (approx. 30 °C). Use a mild dish soap and clean soft brush or sponge.
3. Rinse the parts with cold water (approx. 20 °C) for 10 to 15 seconds.
4. Allow to completely dry after washing.

or

Wash in dishwasher (maximum once per day)

1. Place the disassembled parts, except for the tubing and the Symphony® cap, on the top rack or in the cutlery section in an orientation that prevents the parts from collecting water. Use a commercially available dishwashing detergent.
2. Rinse the yellow membrane under running water after dishwashing.
3. Allow to completely air dry after washing.

i Information

If using the dishwasher, parts may become discolored. This will not impact the functioning of the parts.

Drying

Allow to air dry on a clean, dry dish towel or paper towel.

7.3 Disinfecting

Refer to figure 4 on Chapter 1.

Disinfect on stovetop (before first use and once per day after use)

1. Boil the washed parts for at least 5 minutes. Make sure the parts are fully immersed in water and that are not in direct contact to the bottom of the pot.
2. Allow to completely air dry after disinfecting.

Drying

1. Allow to air dry on a clean, dry dish towel or paper towel.
2. Put the clean and dry parts in a clean storage bag or a clean environment.
Do not store parts in an airtight container/bag if moist. It is important that all residual moisture dries.

8. Assembling your Hands-free Collection Cups

Only use clean and dry parts for assembly. Refer to **chapter 7** for cleaning.

8.1 Selecting the correct breast shield size

Make sure to use the correct breast shield size. It is an essential component of effective pumping, helping to optimize milk flow.

Refer to figure 5 on Chapter 1.

1. 5-01 Select the breast shield size by measuring the diameter of your nipple.
2. 5-02 Select the breast shield size that is slightly larger than your nipple.
24mm size breast shields are included in the pack. 21mm and 27mm size breast shields are available for separate purchase. See **chapter 11** for ordering information.

8.2 Assembling the Hands-free Collection Cups

Only use clean and dry parts for assembly. Refer to **chapter 7** for cleaning.

Parts needed:

- Tubing
- Symphony® cap
- Breast shield
- Yellow membrane
- Outer shell

CAUTION

To prevent contamination of your milk:

- Wash hands thoroughly with soap and water before touching the breast pump, parts of the Hands-free Collection Cups and breasts.
- Dry your hands with a clean towel or a single-use paper towel.
- Avoid touching the inside of the Hands-free Collection Cups components.

To make sure that your breast pump works properly and safely:

- Check the Hands-free Collection Cups' components for wear or damage before use. Replace if necessary.
- Always inspect all parts prior to use for cleanliness. If dirty, clean according to **chapter 7**.
- Only use genuine Medela parts. See **chapter 11** for details.
- **To prevent damage to the breast pump, all components must be completely dry before use.**

Refer to figure  6 on Chapter 1.

1.  6-01 Wash hands thoroughly with soap and water before touching the breast pump, parts of the Hands-free Collection Cups and breasts. Dry your hands with a clean towel or a single-use paper towel.
2.  6-02 First assemble the valve of the yellow membrane on the front of the breast shield.
3.  6-03 Align the disc of the yellow membrane on the designated circular slot using the alignment tab at the top to ensure that it is at the right place. Press around the edges to make sure it is fully assembled.
4.  6-04 Insert the pouring spout into the opening on the shield.
5.  6-05 Push the outer shell and breast shield together tightly, ensuring it is fully sealed all the way around until you hear a snapping sound.
6.  6-06 Connect the flat ends of the tubing into the tubing connection port on the collection cups.

8.3 Preparing for pumping

Refer to figure  7 on Chapter 1.

1. Assemble the Hands-free Collection Cups as described in **chapter 8.2**.
 -  7-01 Connect the round end of the tubing to the Symphony® cap.
 -  7-02 Open the lid of the Symphony® breast pump: Press the oval button on the top of the Symphony® breast pump and lift the handle.
2.  7-03 Position the Symphony® cap firmly on the breast pump membrane to form a seal.
 -  7-04 To confirm the cap is securely engaged, gently press down on it until you hear a distinct “click.” Ensure the cap lies flush with the surface to verify it is properly secured.
3.  7-05 Centre the tubing connector in the recess in the pump housing.
4.  7-06 Close the lid of the Symphony® pump.

8.4 Placing the Hands-free Collection Cups within your bra

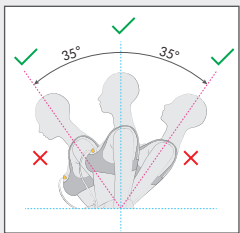
Refer to figure  8 on Chapter 1.

1.  8-01 Ensure that the nipple is centered in the tunnel of the breast shield.
2.  8-02 The Hands-free Collection Cups can be worn under your regular nursing bra. Your bra will hold the Hands-free Collection Cups in place. If you find it uncomfortable or painful during use, try switching to a looser, more stretchy nursing bra.
3.  8-03 The tubing can be connected facing inwards (towards the breasts) or outwards (towards the arms) based on personal preference. Ensure that the tubing is not pinched or bent during use.
4.  8-04 The Hands-free Collection Cups can be used in a vertical or slightly tilted position for better visibility of the nipple and milk ejection. The scale on the Hands-free Collection Cups give you an approximate indication of the volume during pumping.

i Information

14

- The pump is not designed to be used while lying down or sleeping. However, you can slightly lean forward or backward during your pumping session.
- The maximum volume/capacity for the collection cup is 150 ml/5 oz. When this limit is reached, transfer the milk to a bottle before continuing to express.

9. Handling of breast milk

9.1 Removing the Hands-free Collection Cups from your bra

i Information

To avoid milk spilling out of the tunnel, when finished, keep the pump running and slightly lean forward to ensure all the milk inside the breast shield tunnel empties into the collection cups.

Refer to figure 9 on Chapter 1.

1. 9-01 While the pump is still running, slightly lean forward and ensure all milk from the nipple tunnel enters the collection container. Turn off pump.
2. 9-02 Detach the tubing from the Hands-free Collection Cups.
3. 9-03 Carefully remove the Hands-free Collection Cups from your bra.

9.2 Pouring milk from the Hands-free Collection Cups

Refer to figure 9 on Chapter 1.

1. 9-04 Place the Hands-free Collection Cups in an **upright position** on a surface where they will not be easily knocked over.
2. 9-05 **With the rounded outer shell facing down**, pour breast milk through the pouring spout into a bottle or container for storage. **Do not use the Hands-free Collection Cups to store milk. Do not store assembled collection cups in the refrigerator.**
3. Fully disassemble and clean after each use and disinfect once per day as described in **chapter 6** and **chapter 7**.

9.3 General storage guidelines

For information on storing and thawing your breast milk, visit

www.medela.com/breastfeeding/mums-journey/storing-and-thawing-breast-milk

9.4 Long-term storage

Before long-term storage, disassemble and disinfect all parts (except for the tubing and the Symphony® cap).

Do not store the Hands-free Collection Cups in assembled state over an extended period of time.

10. Troubleshooting

Problem	Solution
There is low or no suction	<ul style="list-style-type: none">• Make sure that all the Hands-free Collection Cups components are clean and dry and that connections are secure.• While pumping, make sure the breast shields form a complete seal around the breast.• Refer to the Troubleshooting section in your Medela Symphony® breast pump Instructions for Use.

11. Ordering information

Accessories

Article number	Product
Depending on your region, check local website/shops for availability.	21mm Breast shields compatible with Medela Symphony Hands-free Collection Cups* 24mm Breast shields compatible with Medela Symphony Hands-free Collection Cups* 27mm Breast shields compatible with Medela Symphony Hands-free Collection Cups*
	Symphony® cap compatible with Medela Symphony® breast pump

* If the result of expression is unsatisfactory or expression is painful, contact your lactation consultant or breast feeding specialist. A different size of breast shield may help make expression more comfortable and effective.

Replacement Parts

Article number	Product
Depending on your region, check local website/shops for availability.	Tubing compatible with Medela Hands-free Collection Cups and Symphony® breast pump
	Membrane compatible with Medela Symphony Hands-free Collection Cups

Information

If you have not resolved the problem or in case of difficulties in finding your desired accessory or replacement of lost or defective parts, or any other further questions, please contact Medela customer service/rental agency.

For contact data, visit www.medela.com, select your country.

For any other Medela products information, visit www.medela.com.

12. Warranty

Find information on the international warranty on www.medela.com/ewarranty.

13. Disposal

Breast shields, outer shells, tubing, Symphony® cap and membranes

The parts are made of plastics that are not harmful to the environment when disposed of as household waste. Recycle or dispose of according to local regulations.

14. Meaning of symbols

The following tables explain the meaning of the symbols found on the product packaging/labeling.

	General safety alert symbol, points to information related to safety.		Identifies the manufacturer.
16 	Indicates the date of manufacturing.		Indicates the part number of the product.
	Read and follow the Instructions for Use.		Contains fragile goods. Handle with care.
	Keep away from sunlight.		Keep away from rain. Keep in dry conditions.
	Indicates that the package is capable of being recycled.		Indicates the lot/batch number.
	Indicates the material from which an item is made.		Importer.
	Indicates compliance with the EU requirements concerning medical devices.		Indicates that the packaging contains products intended to come in contact with food according to regulation (EC) No 1935/2004.
	Authorized representative in the European Community / European Union.		Indicates a medical device.
	Indicates a carrier that contains unique device identifier information.		Indicates that an item can technically be recycled according to the French AGECE law.
	Indicates that an item can technically be recycled according to the Royal Decree 1055/2022 (Spain).		

15. Technical specifications

Materials touching skin or coming in contact with milk

- Breast shield : Polypropylene
- Outer shell : Polypropylene, Thermoplastic Elastomer
- Membrane : Silicone
- Tubing : Polyvinyl Chloride (PVC), Acrylonitrile Butadiene Styrene (ABS), Polybutylene Terephthalate (PBT)

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Medela Medical AB